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EN Instructions for use

BSS DISTRA-SOL
Balanced salt solution

Sterile, physiological irrigation solution
for ophthalmological procedures

Composition
1 ml solution contains:

Sodium chloride	6.40 mg
Potassium chloride	0.75 mg
Calcium chloride 2 H ₂ O	0.48 mg
Magnesium chloride 6 H ₂ O	0.30 mg
Sodium acetate 3 H ₂ O	3.90 mg
Sodium citrate 2 H ₂ O	1.70 mg

Excipients:
Water for injection, hydrochloric acid
(for pH adjustment)
No buffered solution
(according to EN ISO 16671)

Intended purpose
For the irrigation of the anterior chamber
during cataract surgery and for other
intraocular or extraocular procedures,
such as therapeutic or diagnostic
measures, where irrigation or retention
of moisture is required.

Side-effects
Where there is pre-existing pathological
damage to the corneal endothelium,
any intraocular intervention – including
intraocular irrigation – can lead to
bullous keratopathy which is character-
ized by corneal oedema or clouding.

After the surgical procedure, inflammatory
processes, including endophthalmitis,
can occur. No connection with the
use of BSS DISTRA-SOL has yet been
demonstrated.

Dosage recommendations
The dosage is determined by the
particular circumstances dictated by
the operative, therapeutic or diagnostic
procedure and should be decided by
the doctor on a case-by-case basis.

Method of administration
To irrigate the eye for the purpose of
surgical procedures as well as diagnostic
and therapeutic measures that require
irrigation or corneal wetting.

An infusion set "P" (pressure-resistant)
with built-in air vent should be used.
Bottles do not have added venting,
which is why the air vent on the infusion
set must be opened. The instruction
for use from the manufacturer of the
infusion set must be always followed.
The infusion set should be fitted onto
the bottle as follows (aseptic conditions):

1. Remove the protective cap from the
adapter on the bottle.
 2. Desinfect the rubber stopper on the
bottle.
 3. Push the puncture spike of the infusion
set into the rubber stopper.
- Note: The fluid should already be
flowing before use, in order to ensure
that there is no air in the tubing.
- The period of use is limited by the
duration of the procedure. One bottle
of BSS DISTRA-SOL is intended for one
patient only.

References in the literature reveal that
the osmolality of the aqueous humour
is higher in diabetics and intraoperative
lens changes can arise. Since BSS
DISTRA-SOL is iso-osmotic with normal
aqueous humour, it should be used
with caution in diabetics undergoing a
vitrectomy. In these cases, it is advi-
sable to match the osmolality to that
of the aqueous humour.

Special instructions
– Use only for eye irrigation and for eye
surgery, not for injection or infusion.

- Only use solution if it is completely
clear and no minute particles are
visible.
- Before use, check that the container
is intact. Do not use, if sterile packaging
was damaged.
- Use the solution immediately after
opening the container.
- BSS DISTRA-SOL contains no preser-
vatives, therefore, do not
store opened containers for further
use. The medical device is intended
solely for single use and is not suitable
for reprocessing.
- Do not use BSS DISTRA-SOL as solvent
for other medicinal products.
- The expiry date relates to an unopened
product in its closed and properly stored
packaging. Do not use beyond the
expiry date given on the packaging.
- Any serious incident that has occurred
in relation to the device should be
reported to the manufacturer /
distributor and the competent
authority of the Member State in
which the user and/or patient is
established.
- Keep out of the reach and sight of
children.

Storage
No special requirements.

Presentation
PP bottle with 250 ml and 500 ml
Not all pack sizes may be marketed.

Date of revision of the text
2024-07

Manufacturer
Serumwerk Bernburg AG
Hallesche Landstr. 105b
06406 Bernburg, Germany

Explanation of the symbols	
	Consult instructions for use
	Do not use if package is damaged
	Do not re-use
	Sterilized using steam
	Single sterile barrier system
	Article number
	Batchcode
	Manufacturing date
	Use-by-date
	Manufacturer
	Medical Device
	Unique Device Identifier

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BSS DISTRA-SOL
Balanced salt solution

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